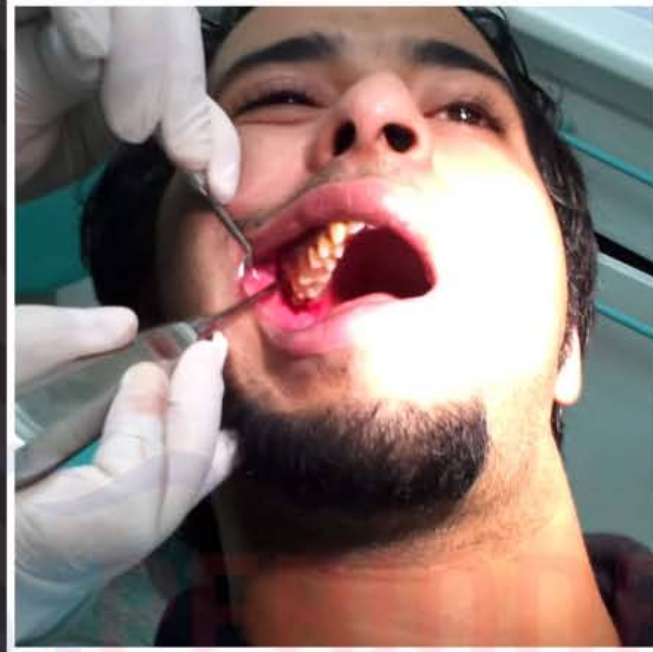
ROOT ELEVATORS

FUNCTION OF ROOT ELEVATORS IN SURGERY

To Loosen Tooth or Root From Bony Socket Prior to Placement of The Extraction Forceps.

HANDLES FOR ROOT ELEVATORS POINTS A, B, C, D, E, F, G, H, I

FUNCTION OF ROOT ELEVATORS IN SURGERY

To Loosen Tooth or Root From Bony Socket Prior to Placement of The Extraction Forceps.

Dental Root Elevators

To look at one of these instruments you might mistake it for a narrow screwdriver.

That's because just like one an elevator has a handle and then a specially designed "blade" or tip portion.

How they're used.

The tip of an elevator is designed to be wedged into the ligament space between a tooth and its surrounding bone (as shown in our animation).

As the elevator is forced into and twisted around in this space, the tooth is rocked around and pressed against the walls of its socket.

This helps to both expand the shape of the socket and separate the tooth from its ligament.

As this work is continued, the tooth gradually becomes more and more mobile.

In some cases, the dentist may be able to completely remove the tooth just using their elevator.

If not, they will choose a point where they feel they have accomplished as much as they can with it and will then switch to using extraction forceps.

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Apexo Root Elevators

FUNCTION: To loosen tooth or root from bony socket prior to placement of the extraction forceps.

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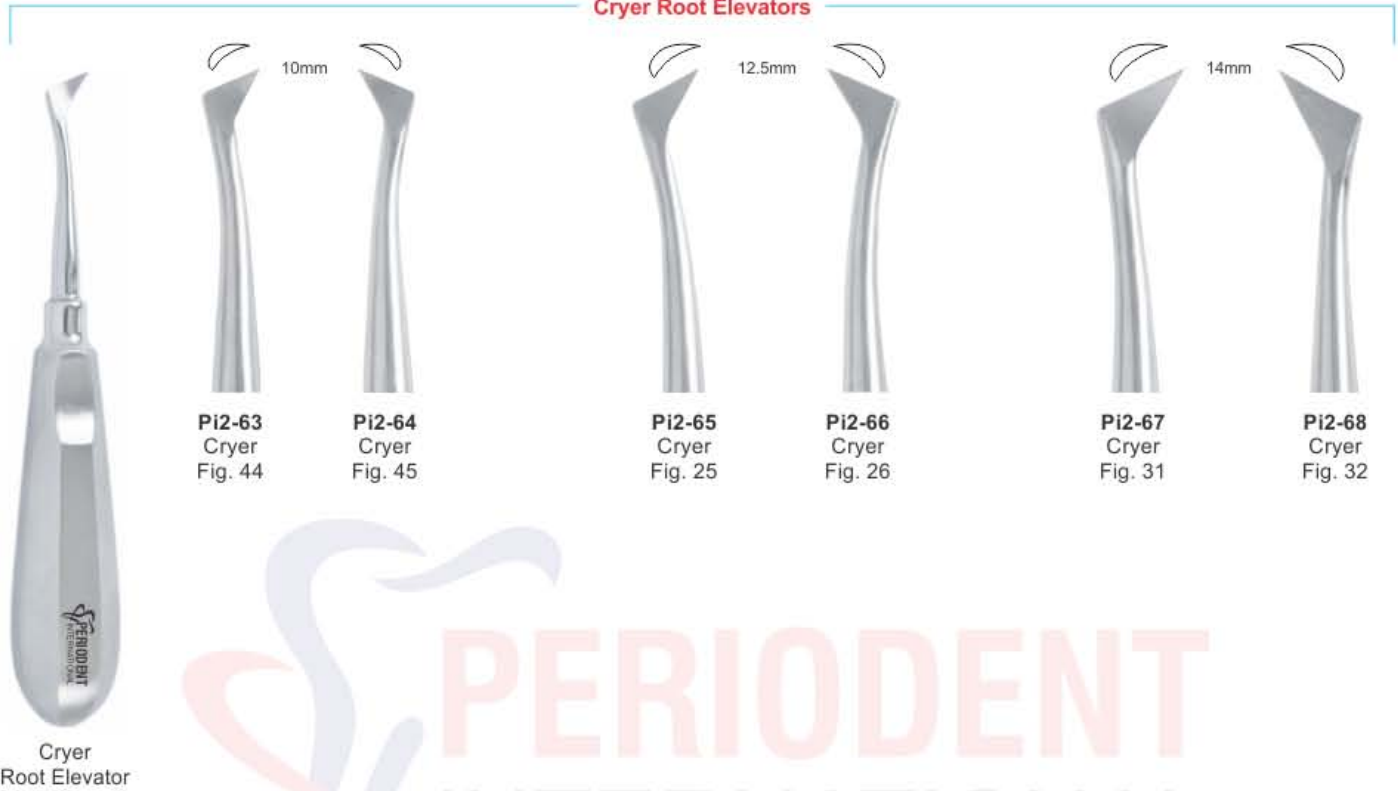
Friedmann Roots Elevator

Heidbrink Root Elevators

Pi2-84 Heidbrink No.1	Pi2-85 Heidbrink No.3 (Left)	Pi2-86 Heidbrink No.4 (Right)

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

Function of Root Elevators: To loosen tooth or root from bony socket prior to placement of the extraction forceps.

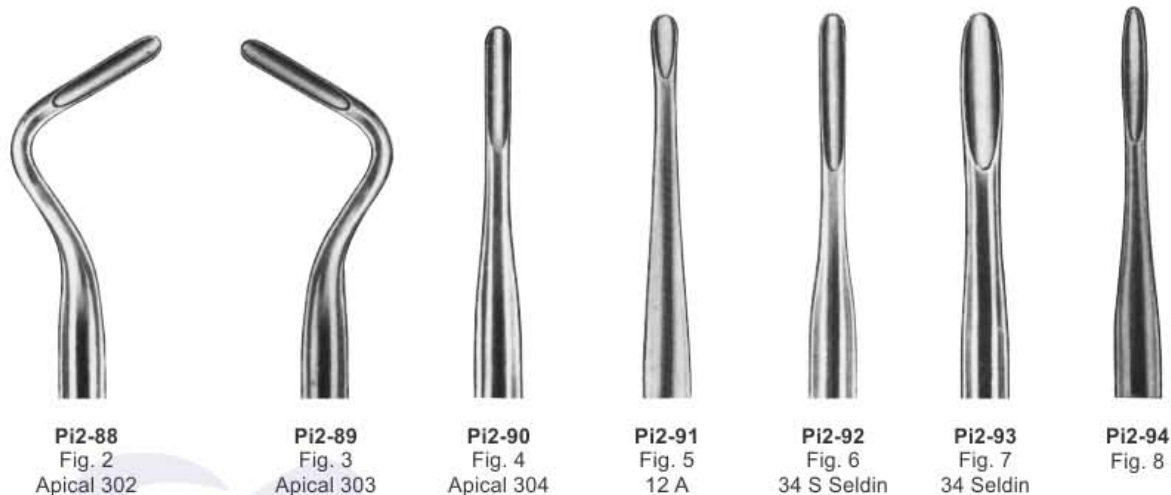Pi2-87
Fig. 1
Apical 301

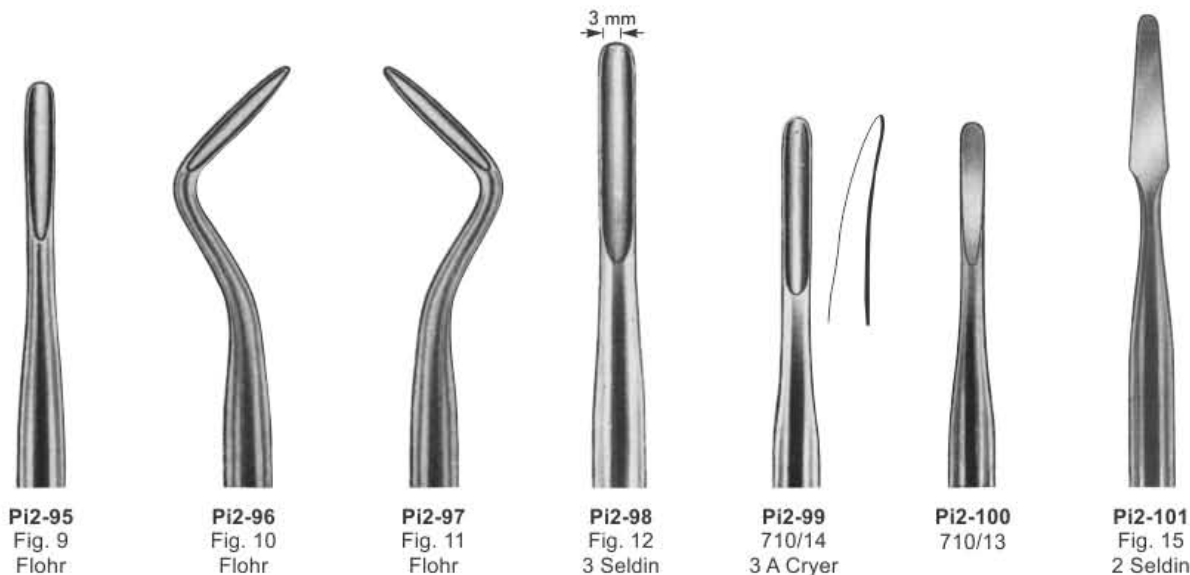Function of Root Elevators: To loosen tooth or root from bony socket prior to placement of the extraction forceps.

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Root Pick Elevator With Gold Flexible Points

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Pi2-157 Curved 3mm	Pi2-158 Straight 5mm	Pi2-159 Curved 5mm	Pi2-160 Straight 2mm	Pi2-161 Curved 2mm	Pi2-162 Straight 4mm	Pi2-163 Curved 4mm
---------------------------------	-----------------------------------	---------------------------------	-----------------------------------	---------------------------------	-----------------------------------	---------------------------------

LUXATING ELEVATORS

Upper root elevator with helicoidal tip

Upper root elevator with helicoidal tip

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Pi2-220
192/3

Coupland Elevators

Coupland root elevators set
3mm, 3.5mm & 4mm tip width
Sand Blasted no slip handle
Surgical Stainless Steel
Sterilized in all Commercial
Sterilizers

1C Elevator Coupland 3mm

Pi2-221

Pi2-222

Pi2-223

Coupland

Pi2-224
Fig. 1
190/15

Pi2-225
Fig. 2
190/16

Pi2-226
Fig. 3
190/17

universal mini-elevator for fragments

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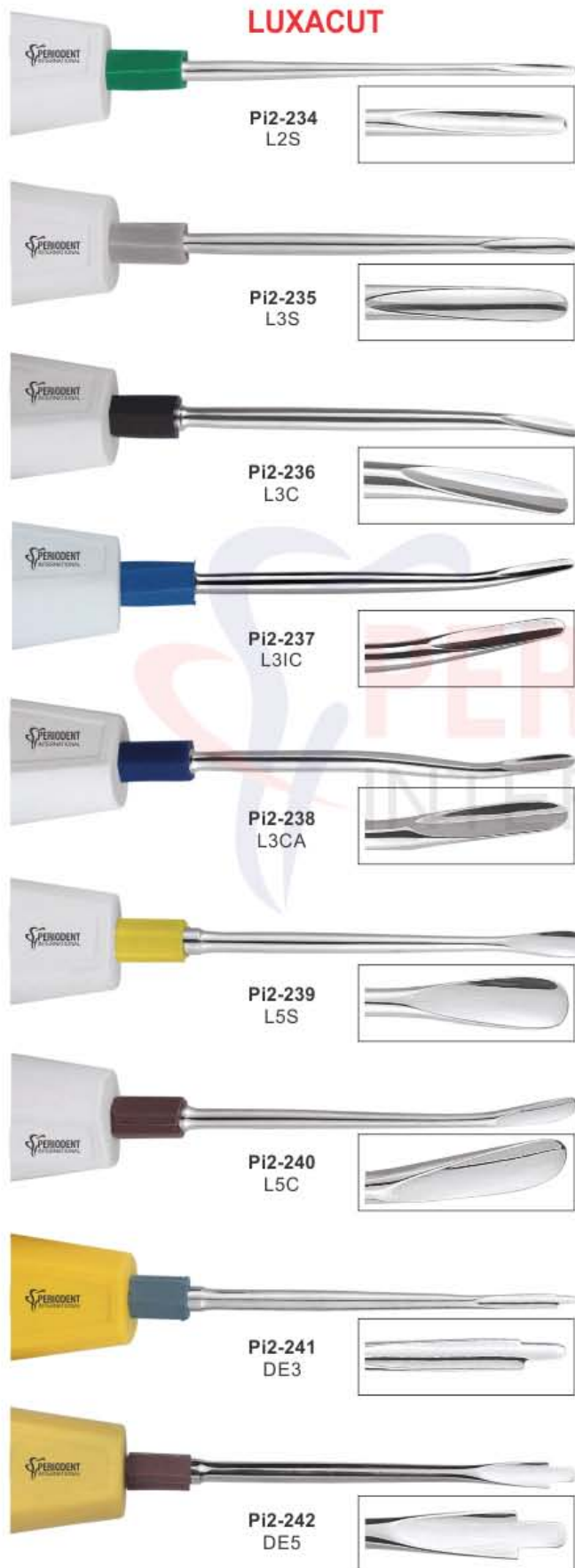
LUXACUT

Luxacut Luxation Instrument

Over recent years, there has been an increased demand for reduced trauma during extraction procedures. The use of elevators has become widespread for the loosening of teeth before the application of forceps. Luxation instruments tips are more finely tapered than elevators tips. Their characteristics are a combination of a Perioste and an elevator. The sharp tip enables more efficient cutting of the periodontal ligament, whilst the tapering shaft assists with controlled expansion of the bony socket. The extraction operation may be performed with the minimum amount of tissue damage. Healing is therefore quicker and the operation is less distressing for the patient and surgical team alike.


The Luxacut Kit comprises of seven Luxation instruments, a straight, curved, inverted curved & contra blade version in Three different widths. Also available in dual edge blades

- PI2-234-L2S Straight Blade- Green-2 mm- Apical and/or Interproximal
- PI2-236-L3C Curved Blade- Black- 3 mm General use/Starter
- PI2-235-L3S Straight Blade- Grey- 3 mm Interproximal
- PI2-238-L3CA Contra Angle- Blue- 3 mm Lingual and/or distal-Molars
- PI2-237-L3IC Inverted Curved- Light Blue- 3 mm- General, Lingual and/or Distal
- PI2-240-L5C Curved Blade- Brown- 5 mm- Large molar roots- General use
- PI2-239-L5S Straight Blade- Yellow- 5 mm- Interproximal
- PI2-241-DE3 dual edge/straight grey 3 mm/1.5 mm
- PI2-242-DE5 dual edge/straight brown 5 mm Deep fractured/Decayed Roots

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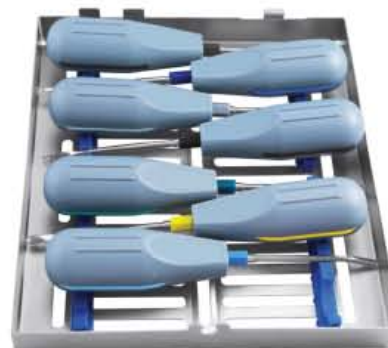Luxating Elevators Set of 11 Pcs Pi2-245

This section includes the Luxating Elevator set with Stainless Steel Handle.
Long & light weight handle design provide excellent smooth grip with controlled force transfer.
The working procedure is similar as mentioned in the previous section.

Pi2-246
Luxating Elevators
1.5mm, Straight

Pi2-247
Luxating Elevators
2mm, Straight

Pi2-248
Luxating Elevators
2mm, Curved

Pi2-249
Luxating Elevators
3mm, Straight

Pi2-250
Luxating Elevators
3mm, Curved

Pi2-251
Luxating Elevators
3mm, Double Curved

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FUNCTION: To Loosen Tooth or Root From Bony Socket Prior to Placement of The Extraction Forceps.

Pi2-260 LUXATOR SET OF 4 PCS STRAIGHT

Pi2-265 LUXATOR SET OF 4 PCS CURVED

APICAL ROOT ELEVATORS

Pi2-270
Fig. 1, 3mm
Frideman
Straight

Pi2-271
Fig. 2
Frideman
Left

Pi2-272
Fig. 3
Frideman
Right

Pi2-273
Fig. 304

Pi2-274
Apical 302A

Pi2-275
Apical 303A

Pi2-286 Root Elevators Set of 10 Pcs

Pi2-276
Root Elevator
DL 32
for upper roots
round point.

Pi2-277
Root Elevator
DL 33
for upper roots
round point.

Pi2-278
Bein
DL 34
for upper roots
round point.

Pi2-279
Flohr
Fig. 3
for lower roots

Pi2-280
Flohr
Fig. 4
for lower roots

Pi2-281
Cryer-White
for lower roots,
wide pattern

Pi2-282
Cryer-White
for lower roots,
wide pattern

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Pi2-289
Wisdom
for 28 and 48

Pi2-290 Potts Elevators set of 2 Pcs

These root elevators are used for releasing the ligaments around the neck of the tooth, and to facilitate the extraction. The working end are smooth These are used for impacted upper 3rd Molars without bony obstruction.

Elevator according to Potts
Working Tips are smooth
Smooth Surface Handle
Surgical Stainless Steel
Sterilized in all Commercial Sterilizers

6 Potts Elevator

7 Potts Elevator

Pi2-291

Pi2-292

Pi2-293 Schmeckebier Apexo Elevators Set of 2 Pcs

Schmeckebier Apexo elevators use to loosen the tooth from the periodontal ligament and ease extraction Sand Blasted no slip handle Surgical Stainless Steel Sterilized in all Commercial Sterilizers

Schmeckebier Apexo Elevator L

Schmeckebier Apexo Elevator R

Pi2-294

Pi2-295

Pi2-298 Apical Elevators Set of 2 Pcs

Apical root elevators, right, left For upper and lower root tips Width of tip 3 mm Smooth Surface Handle Surgical Stainless Steel Sterilized in all Commercial Sterilizers

Apical elevator Left, 3mm

Apical elevator Right, 3mm

Pi2-296

Pi2-297

Pi2-299 Seldin Elevators Set of 3 Pcs

Seldin root elevators Smooth Surface Handle Surgical Stainless Steel Sterilized in all Commercial Sterilizers

34 Str. Seldin Elevator, Upper and lower anterior roots

34S Seldin Elevator, Upper & lower anterior roots, sharp cone edge

2S Seldin Elevator, Lower 3rd Molars

Pi2-300

Pi2-301

Pi2-302

Pi2-303 B-Cogswell & Apexo Elevators Set of 2 Pcs

Smooth Surface Handle Surgical Stainless Steel Sterilized in all Commercial Sterilizers

301 Apexo Elevator, for displacement of deeply seated roots

B Cogswell Elevator

Pi2-304

Pi2-305

Pi2-308 Medium Cryer Root Elevators set of 4 Pcs

Left & Right, with medium working points Working Tips are smooth Smooth Surface Handle Surgical Stainless Steel Sterilized in all Commercial Sterilizers

21Mini Cryer Elevator, Right

22Mini Cryer Elevator, Left

Pi2-306

Pi2-307

44 Cryer Root Elevator, Right

44 Cryer Root Elevator, Left

Pi2-309

Pi2-310

Pi2-311 Large Cryer Root Elevators set of 2 pcs

Left & Right, with large working points Working Tips are smooth Smooth Surface Handle Surgical Stainless Steel Sterilized in all Commercial Sterilizers

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Pi2-314 Posterior Flared Tip Elevators set of 4 Pcs

This section includes a comprehensive set of Posterior Flared Tip Elevators. These elevators used to loosen the tooth from the periodontal ligament and ease extraction, particularly used for Upper Posterior Roots and Upper 3rd Molar.

E46 & E46R: Mesial & Distal with 2.75mm, Smooth working tips
E77 & E77R: Mesial & Distal with 3.75mm, Serrated working tips

Surgical Stainless Steel Hollow handle,
very light weight Completely Sterilizable

Pi2-315 46R Elavator, Smooth, Mesial, 2.75mm
Pi2-316 46 Elavator, Smooth, Distal, 2.75mm
Pi2-317 77R Elavator, Serrated, Mesial, 3.75mm
Pi2-318 77 Elavator, Serrated, Distal, 3.75mm

Pi2-319 Lindo-Levien Elevators Set of 4 Pcs

These root elevators are used for releasing the ligaments around the neck of the tooth, and to facilitate the extraction. The working end serration can be very helpful during the initial phase of extraction.


Elevator according to Lindo-Levien
Working part straight with serration
Width 2.2mm 3.0mm or 4.0mm
Smooth Surface Handle
Surgical Stainless Steel
Sterilized in all Commercial Sterilizers

Pi2-320 Lindo-Levien Elevators 3mm, Straight
Pi2-321 Lindo-Levien Elevators 3mm, Curved
Pi2-322 Lindo-Levien Elevators 4mm, Straight
Pi2-323 Lindo-Levien Elevators 4mm, Curved

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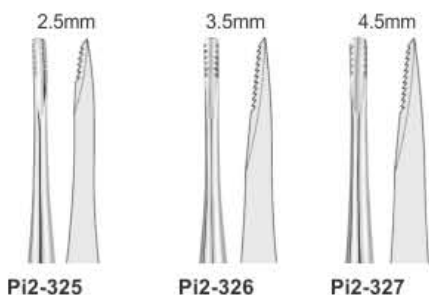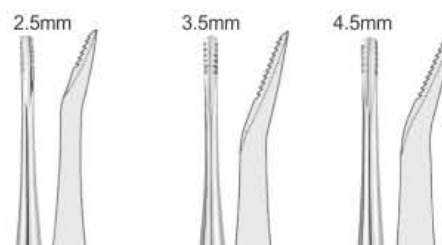
Laxutor Root Elevators Set of 10 Pcs Pi2-332

Long Type
Laxutor
Root Elevator

Pi2-333
Laxutor
1.5mm
Str.

Pi2-334
Laxutor
2mm
Str.

Pi2-335
Laxutor
3mm
Str.

Pi2-336
Laxutor
5mm
Str.

Pi2-337
Laxutor
3mm
Cvd.

FUNCTION:

To loosen tooth or root from bony socket prior to placement of the extraction forceps.

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2mm 3mm 4mm

Pi2-391 Bein DL 32 for upper roots round point.

Pi2-392 Bein DL 33 for upper roots round point.

Pi2-393 Bein DL 34 for upper roots round point.

Pi2-394 Cryer 27 (Left)

Pi2-395 Cryer 28 (Right)

Pi2-396 Flohr 3mm

Pi2-397 Flohr 3mm

Pi2-398 Root Elevator Set of 7 Pcs

ROOT ELEVATORS SET OF 7 PCS

2mm 3mm 4mm

Pi2-399 Bein DL 32 for upper roots round point.

Pi2-400 Bein DL 33 for upper roots round point.

Pi2-401 Bein DL 34 for upper roots round point.

Pi2-402 Flohr Fig. 3 for lower roots

Pi2-403 Flohr Fig. 4 for lower roots

Pi2-404 Cryer-White for lower roots, wide pattern

Pi2-405 Cryer-White for lower roots, wide pattern

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Bernard Elevators -Syndesmotomes

Set of 5 Pcs

Bernard Root Elevators are particularly suitable for the excision of fractured roots or root fragments.

They represent a combination of root tip elevators and luxating elevators.

The inner side is concave and the laterals edges are for cutting. This allow for rotatory movements, and when applying slight pressure, the ligaments can be separated and the remaining root fragments can be loosened.

This section includes a set of five Bernard Root Elevators, including Left, Right, Straight, Mesial & Distal form.

All elevators are made from Surgical Stainless Steel Hollow handle, light weight Completely Sterilizable Bernard Root elevators also know as Syndesmotomes.

They can also be use as Periostome or to elevate Small Roots

PI2-419
Distal Spade Root Tip Elevator

PI2-420
Mesial Spade Root Tip Elevator

PI2-421
Straight Spade Root Tip Elevator

PI2-422
Right Spade Root Tip Elevator

PI2-423
Left Spade Root Tip Elevator

BAYONET ELEVATORS SET OF 7 PCS

PI2-425

PI2-426

PI2-427

PI2-428

PI2-429

PI2-430

PI2-431

PI2-432

PDL Elevators Set of 7 Pcs

PI2-433

PDL Elevators Set of 7 Pcs Color Coated

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Area: mesial

Pi2-434 PDL Elevators Set of 10 Pcs

			Pi2-435 Luxa-Tool, straight, 3.0 mm,
			Pi2-436 Luxa-Tool, curved, 3.0 mm,
			Pi2-437 Proximator, curved, 2.5 mm, distal elevator,
			Pi2-438 Proximator, curved, 2.5 mm, mesial elevator,
			Pi2-439 Proximator, curved, 4.5 mm, distal elevator,
			Pi2-440 Proximator, curved, 4.5 mm, mesial elevator,
			Pi2-441 Syndesmotome, curved, 4.0 mm, for distal access
			Pi2-442 Syndesmotome, curved, 4.0 mm, for mesial access
			Pi2-443 Syndesmotome, curved left, 4.0 mm, for lingual/buccal luxation,
			Pi2-444 Syndesmotome, curved right, 4.0 mm, for lingual/buccal luxation,

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